


Listen EVERYWHERE Assisted Hearing


1. Connect to the ListenEverywhere WiFi Network*
Go to Settings > WiFi >
Select the ListenEverywhere wireless network

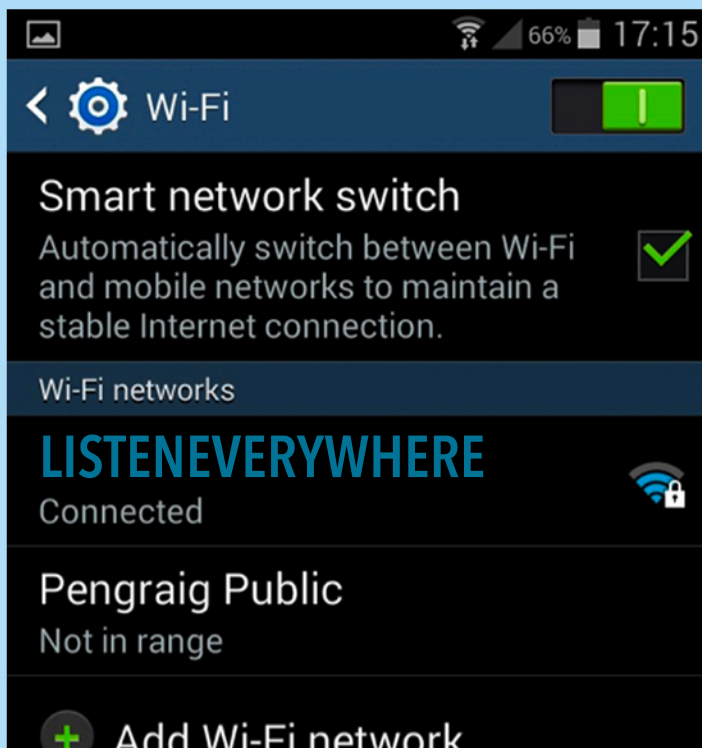

Scan to connect to the ListenEverywhere WiFi Network




2. Open the ListenEverywhere app on your phone.*

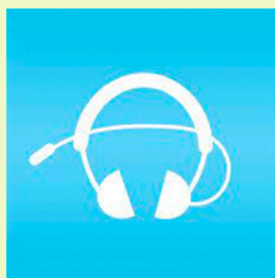

3. Press the Play button



4. Listen through your headphones or Bluetooth enabled hearing aids connected to your phone.*

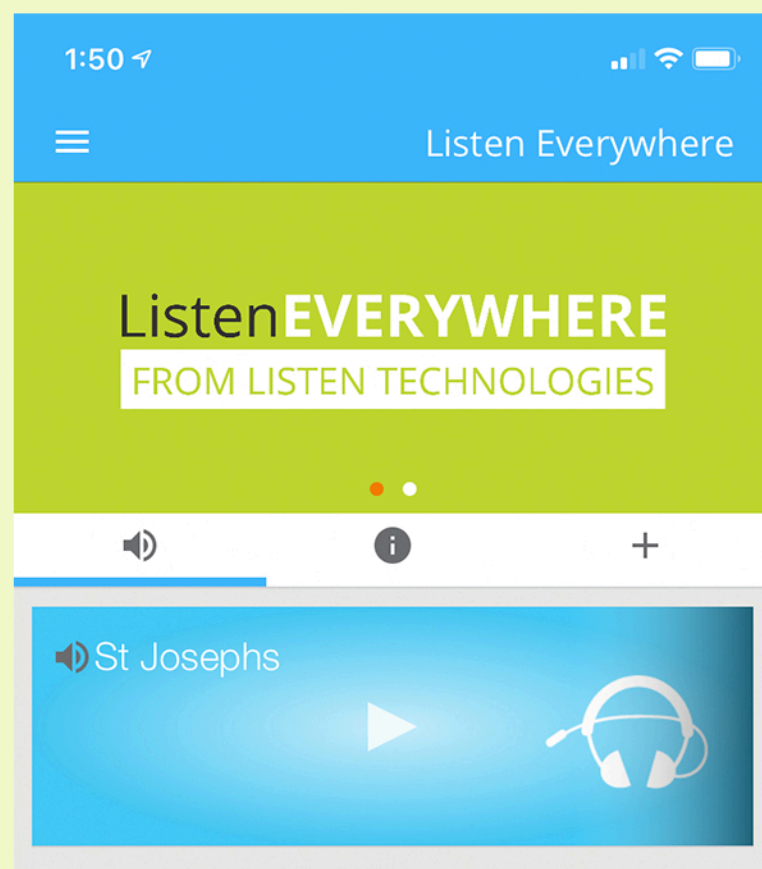

Scan to Open/Download the ListenEverywhere app on your phone



* If you have not already done so:

1. Turn on WiFi
2. Turn on Bluetooth
3. Download the Listen EVERYWHERE app to your phone.